WHITE | BEAUTY PROFESSIONAL **EASY LAMP PLUS**

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3.Product Features..... 4

4.Technical Specification 5

5.Description of product structure and components 5

6.Machine handling instruction 6

7.Simple troubleshooting guide 8

8.Disposal 8

9.Warranty 8

10.Declaration of conformity..... 10

USER MANUAL

1. SAFETY PRECAUTION

PLEASE NOTE: Prior to installation and start – up of the unit, carefully read the instruction provided herein as with all technical devices, the proper function and safe operation of this unit depend on the user's compliance with the safety recommendation presented in these operating instructions.

CAUTION: It is important to observe the following precautions:
Do not start the light directly onto or toward unprotected gingival or skin.

CAUTION: Persons with a history of photosensitivity or who are using photosensitizing drugs should not be exposed to the light from this unit.

CAUTION: This equipment is not suitable for use in presence of flammable anesthetic mixtures with air or nitrous oxide.

CAUTION: Please do not look into emitted light by the machine when it is use or it will hurt your eyes, please be sure to wear goggles.

CAUTION: The bend angle of the arm should not exceed 90 degrees to prevent undesirable noise in the future.

2. STOCK AND TRANSPORT

1. In order to ensure the safety of the product during transportation, the product should be handled with care, avoiding stacking up more than five layers of boxes and avoiding to let any liquid to pour into the boxes.
2. Store in -10°C ~40°C, avoiding any corrosion, and keep in a clean well -ventilated environment with humidity in the range of 30%----- to 80% indoor environment.

3. PRODUCT FEATURES:

- The bleaching system is simple to use as it is controlled by a microcomputer inside the device.
- Its output blue light wavelength is 460-490 nm and it can irradiate all surfaces of the teeth effectively.
- The wavelength is specifically tuned to accelerate the oxidation process in the interprismatic spaces of the enamel. The oxygen penetrates the interprismatic space to reach the surface of dentine, then “cutting” the stains that are present in the form of long chains. After the “cutting” the stain become invisible.
- The Led head unit has a unique design, the light source has a wide range, and the light radiates at a perfect angle and with optimal uniformity.

WHITE BEAUTY PROFESSIONAL EASY LAMP PLUS

- The distance between the LED head unit and the cheek retractor will make sure that the LED head unit keeps clean, preventing cross infection and meeting high hygiene standards

4. TECHNICAL SPECIFICATION

- Dimension of frame: height 1420 mm, Arm length: 1250mm (max), wide: 630mm
- Wavelength range: 460nm - 490nm
- Power: 40 V
- Output: 4.7 V
- Operating voltage: 100V-240 V AC, 50/60 Hz
- Density of the blue light: 200-300 mw/cm²
- Whitening time; 10-15 minutes for one time, two or three for a treatment
- Weight: 18.5 kg

5. DESCRIPTION OF PRODUCT STRUCTURE AND COMPONENTS


Components instruction:

- 1- Horse
- 2- Shoring
- 3- Power box
- 4- Rotational arm
- 5- Handle
- 6- Main Frame
- 7- Power Wire
- 8- Screen
- 9- Time button
- 10- Power button
- 11- A-B Group button
- 12- Down button
- 13- Up button


WHITE BEAUTY PROFESSIONAL EASY LAMP PLUS

- 14- On/Of button
- 15- LED Lights (Group A)
- 16- LED Lights (Group B)

6. MACHINE HANDLING INSTRUCTION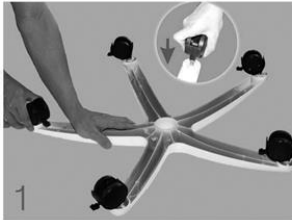


1. Install all 5 truckels into the horse;

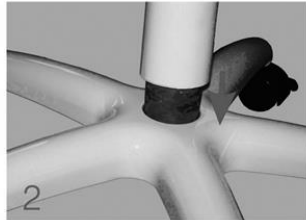2. Insert the bottom of the stud into the central hole of the horse;

3. Get the screw across the piece of round iron, wind the screw into the bottom of the stud, then screw down it with a wrench in the direction of the hour hand;

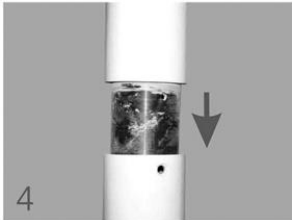
4. Connect the two pieces of the piler and tie up with the screw.

5. Screw down the tie-in of the rotational arm into the arm in the direction of the hour hand;

6. Connect the plug of the rotational arm and the socket of the stud;

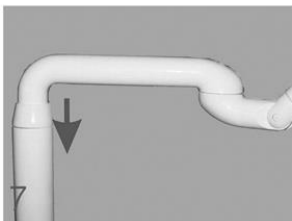7. Infix the connector of the rotational arm into the inner hole of the stud;

8. Infix the connector of the rotational arm into the inner hole of the stud;

9. Insert the connector of the mainframe into the inner hole of the rotational arm, fix the screw with a screwdriver that out-fit with the Bleaching system.


1. The bleaching system should be assembled before using. Please open the package. Take out the parts from the package, throw away the protecting film, assemble the equipment and check whether all the connecting wires are connected all right.
2. Put the attaching plug of the bleaching system into the power socket, and then open the power switch of the power box (ON), at the same time the screen of the machine will turn on, and it will show every parameter before closing the machine last time. See picture 1:

WHITE BEAUTY PROFESSIONAL EASY LAMP PLUS


Picture 1

3. Press the key "TIME": the time on display will shine, at that moment you can adjust the time by the key UP ▲ and DOWN ▼ - when you press the key for one time, it will increase or decrease 1 minute - and the maximum range for the adjustment time is 30 minutes. You can set the whitening time up to your needs. After the setting time, you can press the key "TIME" again to fix the setting time. See picture 2: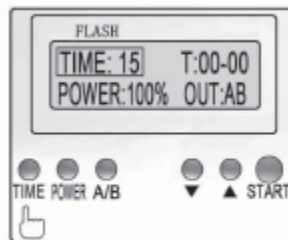


Picture 2

4. Press the Key "POWER": the magnitude of the output power will shine; at that time, you can adjust the key UP ▲ and DOWN ▼ in order to set the magnitude which you need. When you press one time, the output power will increase or decrease by 10%. The range of the output power is from 40% to 100%, you can select every power you want, according to your needs. After the setup, you can press the key "POWER" again to fix the power you set. See picture 3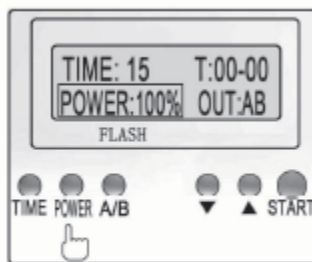


Picture 3

5. Press the key "A/B": the selection of the LED group will shine; at that time, you can adjust using the key UP ▲ and DOWN ▼. There are 3 modes to be chosen: "A" for the upper group; "B" for the lower group; "AB" for both groups. After selection press "A/B" again to confirm. See picture 4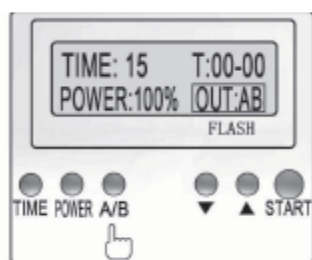

Picture 4

6. Press the key "START", the whitening system will shine, and at that time the whitening system will work according to the setting instructions, with the display showing the remaining working time. When the time is over, the whitening system will close automatically; when you want to stop whitening process, please press the key "START" again (Note: when the bleaching system stops working, the cooling fan inside the machine will go on working 1 minute to protect the LED pipe and the safety system). See picture 5:

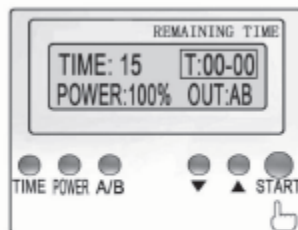Picture 5

7. There is a thermometer guard unit in the bleaching system, when the temperature of the blue light led pipe exceed 60°C, the whitening system will stop working to protect the safety system.

7. SIMPLE TROUBLESHOOTING GUIDE

The sort of the Fault	The reasons of the Fault	Solutions
No display	Switch off	Turn on the switch
No output	No setting of the relevant parameters	Set the parameters properly
Low luminosity	Selection of just on group of LED (only A or Be)	Select both A and B
Low luminosity	Low output power	Set 100% output power
Any other breakdown		Contact our after -sale department

8. DISPOSAL

According to Directive 2012/19/EEC, this symbol indicates that the product should not be disposed as a general waste at the end of its lifespan. The product must be taken to a specialized centre for the separate collection of electrical and electronic equipment according to local regulations.

9. WARRANTY

- The warranty period for this product is 24 months.
- The device and its components must only be used as intended by the manufacturer purpose, as outlined in this Operating instruction. Any other use will void the warranty and the manufacturer is not liable for any damages or damages caused by it.
- If during the warranty period the device fails due to improper operation (mechanical, chemical, thermal, electrical), non-intended use, inappropriate storage etc., warranty will be void and the repair cost will be at the expense of the user.

WHITE BEAUTY PROFESSIONAL EASY LAMP PLUS

- The device should not be used with damaged cables. If any such damages are noticed switch off the device and immediately take it to the service facility.
- The warranty of the device will be forfeited if any repairs or modifications are done by unauthorized personnel and/or non-original spare parts are used.
- Sending for repair to the company service must be done in the original packaging of the product.
- No claims shall be accepted for damages arising from electrical shocks, thunderstorms, non-compliance with electrotechnical safety measures or insufficient protection of patients, staff, other people, animals, plants, and objects from light radiation.

The warranty is forfeited, and no claims shall be accepted for damages because of incorrect or insufficient care and attention for protection during transport, unpacking, moving, handling and storage of the device.

10. DECLARATION OF CONFORMITY

 DECLARATION OF CONFORMITY ACCORDING TO (EU) 2017/745 MEDICAL DEVICE REGULATION	
EU Representative SUNGO Europe B.V. Olympisch Stadion 24, 1076DE Amsterdam, Netherlands SRN: NL-AR-000000247	Manufacturer Name: ZHENGZHOU HUAER ELECTRO-OPTICS TECHNOLOGY CO., LTD Address: 11th floor, Block B, Building 18, Henan International University Science Park (East District) ZhengZhou, Henan, P.R. China
Conformity Assessment Conformity Assessment Procedure Annex II+III of Regulation (EU) 2017/745	Product Information Name: TEETH WHITENING LIGHT Model: N/A GMDN: 31806 Basic UDI-DI: / Classification: Class I, According to Rule 1, Annex VIII, Regulation (EU) 2017/745
Applicable Standards EN ISO 14971: 2019 EN ISO 15223-1: 2016 EN 1041:2008+A1:2013 ISO 10993-1: 2018 EN ISO 10993-5: 2009 EN ISO 10993-10: 2013	Declaration We herewith declare that the above-mentioned products meet the requirements of Medical Device Regulation (EU) 2017/745 and the applicable standards above.
Remark <i>The declaration of conformity is valid in connection with the release technical document CE/MDR-TBAU-01.</i> <i>All the supporting documentation is retained at the premises of the manufacturer.</i> <i>The Declaration of Conformity is exclusively under the sole responsibility of the manufacturer.</i>	Signature: <i>XiaoJun Li</i> Date: <i>2021.6.28</i>  Position: GM Place: Zhengzhou/China

Zhengzhou Huaer Electro-Optics Technology Co., Ltd

Floor 5, D of Building 18, the National University Science park of Henan Province, China

Sungo Europe B.V. Olympisch Stadion 24, 1076 DE Amsterdam, Netherland

Distributed by: MIROMED S.r.l. Via De Gasperi, 17 – 20045 – Lainate (MI) - Italy

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
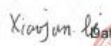 DECLARATION OF CONFORMITY ACCORDING TO (EU) 2017/745 MEDICAL DEVICE REGULATION	
EU Representative SUNGO Europe B.V. Olympisch Stadion 24, 1076DE Amsterdam, Netherlands SRN: NL-AR-000000247	Manufacturer Name: ZHENGZHOU HUAER ELECTRO-OPTICS TECHNOLOGY CO., LTD Address: 11th floor, Block B, Building 18, Henan International University Science Park (East District) ZhengZhou, Henan, P.R. China
Product Information	
Conformity Assessment Conformity Assessment Procedure Annex II+III of Regulation (EU) 2017/745 Applicable Standards EN ISO 14971: 2019 EN ISO 15223-1: 2016 EN 1041:2008+A1:2013 ISO 10993-1: 2018 EN ISO 10993-5: 2009 EN ISO 10993-10: 2013 Remark The declaration of conformity is valid in connection with the release technical document CE/MDR-TBAU-01. All the supporting documentation is retained at the premises of the manufacturer. The Declaration of Conformity is exclusively under the sole responsibility of the manufacturer.	Name: TEETH WHITENING LIGHT Model: N/A GMDN: 31806 Basic UDI-DI: / Classification: Class I, According to Rule 1, Annex VIII, Regulation (EU) 2017/745
Declaration	
We herewith declare that the above-mentioned products meet the requirements of Medical Device Regulation (EU) 2017/745 and the applicable standards above.	
Signature:  Date: 2021-6-29  Position: GM Place: Zhengzhou/China	

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Sungo Europe B.V. Olympish Stadion 24, 1076 DE, Amsterdam, Netherland
DISTRIBUITO DA: MIROMED S.r.l. Via De Gasperi, 17 – 20045 – Lainate (MI) - Italia