

	 MIROMED GROUP s.a. Corso San Gottardo, 54A - 6830 Chiasso - CH Ph: +41 (0)91 960 58 48 info@miromedgroup.ch - www.miromedgroup.ch MIROMED S.R.L. licensee MENTADENT PROFESSIONAL EC REP MIROMED S.R.L. Viale De Gasperi,17 - 20045 Lainate (MI) - ITALY Ph: +39 0293572150 - info@miromed.it - www.miromed.it	
	QUICK MASK BLUE	
White Beauty Professional Quick Mask Blue		
TECHNICAL DATA SHEET		
Class I - Medical Device Regulation 2017/745 (MDR)	Rev. n°	03
	Date	02/07/2024

1. Composition

Di-hema Trimethylhexyl Dicarbamate, Silica, Hema trimellitate Anhydride, Titanium Dioxide, Ethyl trimethylbenzoyl phenylphosphinate, Sodium Saccharin, C.I. 42090.

2. Chemical and Physical properties

Appearance: Blue colored viscous/flowable paste.

Odor: Ester like.

3. Field of use

White Beauty Professional QUICK MASK BLUE is a light cure gum protector for professional use only.

4. Mechanism of action

White Beauty Professional QUICK MASK BLUE is a very fast light cured resin that is used to isolate the gum adjacent to teeth undergoing whitening. Its fluid consistency guarantees exact positioning of the product and excellent sealing of interproximal spaces.

5. Method of use

White Beauty Professional QUICK MASK BLUE is packaged in syringes and can be positioned directly on the gum in the absence of bubbles, using the applicator tip (Flow Tip) always included in the package. After teeth whitening it is extremely easy to remove the product.

6. Controindications

White Beauty Professional QUICK MASK BLUE is contraindicated in patients with history of severe allergic reaction to methacrylate resins.

7. Caution

Read the instruction for use prior to beginning the treatment. Use only as described Keep out of children's reach.

For professional use only.

8. Mechanism of polymerization

Photopolymerization with a lamp with a minimum power of 600 mW/cm² and a wavelength of 400 - 500 nanometers.

9. Storage

Keep stored at 10°C to 24°C.

If it is stored at the refrigerator bring the product to room temperature (between 21°C and 24°C) before use.

Do not keep the product net to products which contains eugenol.

Do not expose *White Beauty Professional QUICK MASK BLUE* to elevated temperatures or intense light.

10. Shelf life

24 months

INFORMATION RESTRICTED TO HEALTHCARE PROFESSIONAL AND PROFESSIONA USERS